SELECT

PROFCARE

Classified as category 1
Conformity Declaration: this product meets the Directive 93/42/ECC on medical devices.

Made in China
SELECT SPORT A/S • Fabriksparken 46 • DK-2600 Glostrup
select-sport.com

UK

Knee Strap - Art. 70357

Function:

- Used for Schlatler/Jumper's knee
- Provides pressure under the knee
- For protection against minor impacts from falling, protection against bruises, abrasions and light burns

Instruction for use:

Place the support as shown on the front of the packaging.

Material:

- 40% Neoprene, 30% Polyester, 10% Rubber, 10% Polypropylene, 10% Polyamide

Important information:

- Do not use on patients with a known sensitivity to any of the product components
- The product is not flameproof, avoid temperature above 120°C/248°F.
- The function of the product must not be changed or compromised
- The product should only be used for its original purpose as described above
- Time of use should not exceed 3-4 hours
- If the product contains latex or rubber and you are allergic to latex/rubber, we advise you to consult a prescriber before using the support
- If you are allergic to heat, we advise you to use a product that does not contain SBR Neoprene

To ensure correct fit, try the product on.

Washing instructions:

1. Close all Velcro
2. Mesh laundry bag is recommended
3. Stretch product while wet and let dry

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